ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER

The intended user should be the patient, a licensed healthcare professional, or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS

DonJoy Lumbostrap 35 is designed to support and stabilize the lumbar spine.

- Stable vertebral fractures of the dorso-lumbar tract
- Post-operative recovery
- Results of vertebral collapse (most common indication), that requires a very long usage (up to 12 months)
- Lumbaritis
- Sacroiliacalgia and lumbar cruralgia
- Spondylolisthesis and lumbar discopathies
- Minor trauma to the lumbar/sacral column and paravertebral lumbar contractures
- Treatment of fracture of transverse processes
- Degenerative lumbar spondiosis
- Spondylosis with or without vertebral listhesis

PERFORMANCE CHARACTERISTICS

- Compression: Providing mild compression of limb or body segment through elastic construction.
- Support: Soft-goof/Semi-rigid designed to restrict motion through elastic or semi-rigid construction.
- Immobilization: Providing immobilization or controlled movement of the body segment.
- Protection: Providing mild protection of the body segment for acute and prophylactic care.

CONTRAINdicATIONS

- Pregnancy and circulation, pulmonary, cardiovascular or skeletal conditions which pose risk to be made worse as a result of compression of the abdomen and/or pressure, unstable, displaced fractures.
- Do not use if you are allergic to any of the materials contained within this product.

WARNING AND PRECAUTIONS

This device is prescribed, adjusted and explained by a healthcare professional. The product is intended for single patient use only.

We recommend that the first application be made under supervision of a healthcare professional. Contact your healthcare professional if you are in doubt about the application.

Do not wear the device near to open flames.

The frequency and duration of use should be determined by your healthcare professional.

Direct contact of the device to very sensitive skin may cause redness of irritation. Immediate medical attention is needed in case of intolerance.

Do not fit too tightly, as this could lead to local pressure symptoms.

Do not apply to wounds.

This product is for single patient use only.

Do not use this device if it was damaged or defective and/or packaging has been opened.

Do not modify the device or use the device other than intended.

The product should be removed after long periods of rest.

Do not use during showering or bathing.

Do not use this product in any way that may damage its integrity.

If patient experiences any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your healthcare professional immediately.

Note: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS

The product is designed to be worn during everyday activities. Must be worn over a T-shirt or light clothing.

Your healthcare professional will explain the application of this back brace and make all adjustments necessary for a proper fit.

Frame adapts to the curvature of the spine. The removable aluminum stays can be swapped out for a more rigid version indicated with a sticker marked R supplied with the brace.

1. Open the lumbar belt and fasten all straps.

2. Place the lumbar support around the waist using the hand loops. Make sure that the lumbar belt is correctly centered over your spine.

3. Close the lumbar belt around your waist.

4. Pull adjustable tension straps forward to adjust the level of compression and support needed.

CLEANING INSTRUCTIONS

The aluminum dorsal frame should be taken out of the brace before washing. Please check you place the stay back in the pocket after washing.

Hook fasteners of the straps and belt should be closed for washing to avoid any damage.

Hand wash the product with lukewarm water (40°) and neutral soap, then rinse it carefully with clean water.

Air dry and keep it away from heat sources. Do not machine dry. Do not dry clean. Do not iron.

Contact your medical/orthopedic technician in case of problems or worn components which need replacement.

COMPOSITION

Polyester, elastane, polyamide, steel, polyethylene, polyurethane, cotton

NOT MADE WITH NATURAL RUBBER LATEX

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

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